

PREempt[®]

One-Step Disinfectant
Cleaner Wipes formulated

with  **AHP**
Patented



Wipes

WHY CHOOSE PREempt™?

- **CLEANER** PREempt™ Wipes are a one-step disinfectant cleaner. It has powerful surfactants, which means it disinfects while it cleans surfaces and equipment, resulting in lower cost and faster results.
- **FASTER** Realistic contact times on a broad spectrum of organisms resulting in user compliance and confidence that disinfection has occurred.
- **RESPONSIBLE** Users are not exposed to VOC (volatile organic compounds). Proven to be essentially non-toxic and non-irritating to eyes and skin.
- **SUSTAINABLE** Hydrogen Peroxide breaks down into water and oxygen, leaving no active residual and reducing environmental impact.

DIRECTIONS FOR USE

Disinfection of Non-Critical, Pre-cleaned Instruments and Medical Devices:

1. Pre-clean heavily soiled areas.
2. Pull towelette from canister and wipe hard, non-porous environmental surfaces.
3. All surfaces must remain visibly wet for 1 minute. Use a 5 minute contact time for TB. Use a 10 minute contact time for fungi.
4. Allow to air dry.

For use in laboratories, clean rooms and other critical environments that require cleaning and surface disinfection. This includes work stations, fume hoods, laboratory counter tops, equipment and other hard non-porous environmental surfaces.

AVAILABLE PRODUCTS

PREempt™ One-Step Disinfectant Cleaner Wipes

DIN: 74559-3

SHELF LIFE: 36 Months

CONCENTRATION OF ACTIVE:
0.5% Hydrogen Peroxide

1. Format Size: 6"x7" – 160ct Wipes
Product Code: 21221
Packaging Specifications: 12x1 Canister

EFFECTIVE AGAINST A BROAD-SPECTRUM OF BACTERIA, *VIRUSES AND FUNGI IN 1 MINUTE!

*Virucidal: 1 Min.

- *Poliovirus Type 1
- * Kills HIV-1 (AIDS virus)
- * Kills Norovirus (Feline Calicivirus as the surrogate)
- *Human Coronavirus
- *This product has demonstrated effectiveness against Influenza A virus and is expected to inactivate all Influenza A viruses including Pandemic 2009 H1N1 Influenza A virus.

Bactericidal: 1 Min.

- *Pseudomonas aeruginosa*
- *Staphylococcus aureus*
- *Salmonella enterica* (formerly known as *Salmonella choleraesuis*)

Antibiotic-Resistant Bactericidal Activity:

- Methicillin-resistant *Staphylococcus aureus* (MRSA)
- Vancomycin-resistant *Enterococcus faecalis* (VRE)
- *Escherichia coli* with extended beta-lactamase resistance (ESBL)

Fungicidal: 10 Min.

- *Trichophyton mentagrophytes*

Tuberculocidal: 5 Min.

- †*Mycobacterium bovis* (BCG)

Broad-Spectrum Non-Food Contact Sanitizing: 30 Sec.

- *Klebsiella pneumoniae*
- *Salmonella enterica* (formerly known as *Salmonella choleraesuis*)
- *Pseudomonas aeruginosa*
- *Staphylococcus aureus*
- *Escherichia coli* O157:H7



PREempt™ One-Step Disinfectant Cleaner Wipes are designed for use in many establishments including compound pharmacy labs, laboratories, clean rooms and critical environments where the requirement for broad-spectrum germicidal performance in short contact time is required.

*Please refer to reference sheet for a complete list of pathogenic organisms, additional features, and claims.

PREempt™ brand products are manufactured by Virox Technologies Inc.

Accelerated Hydrogen Peroxide® and PREempt™ are registered trademarks of Virox Technologies Inc.



For more information on this or our other products, visit:

> virox.com

2770 Coventry Road, Oakville, ON CANADA L6H 6R1
1-800-387-7578 | www.virox.com



TB† stands for BCG (*Mycobacterium bovis*)

Engineering Revolutionary Disinfectants for the War Against Microbes