AHP1750

Indicator Strips

For Use with 7% Concentrate
AHP Products (1:40 Dilution)

Part No. AHP1750

Description

Accelerated Hydrogen Peroxide® (AHP®) Indicator Strips are easy to use dip-and-read reagent strips to confirm the presence of the minimum recommended concentration of hydrogen peroxide, the active ingredient in AHP®, **AHP1750 Test Strips were DESIGNED for use with 7% Surface Disinfectant Concentrates diluted to 1:40 (1750ppm) or 1:128 (500ppm).**

NOTE: AHP1750 Indicator Strips ARE NOT intended for use with 7% AHP Concentrates diluted at 1:16 (0.5% or 500ppm).

Direction for Use

1. Dip the padded end of the indicator test strip into the diluted solution and remove after one second.
2. Do not shake the test strip. Once removed, hold the strip horizontally and start the timer.
3. **For Disinfection (1:40)**, the indicator test strip colour should match the colour block between 35 and 40 seconds to indicate a **PASS**. **For Sanitization (1:128)**, the indicator test strip colour should match the colour block between 110 to 120 seconds (1 minute 50 seconds to 2 minutes) to indicate a **PASS**.

NOTE: Delayed reading of the strip colour may invalidate the results. The strip colour will continue to darken after the read time.

The following table provides an easy comparison of the dilution rates for the AHP products to parts per million (ppm).

Product Concentrate & Dilution Rate	PPM
7% AHP diluted at 1:40	1750 ppm
7% AHP diluted at 1:128	500 ppm

Shelf life

Do not use the strip after the expiration date. Please refer to expiry date on indicator test strip bottle.

Storage

Store the indicator test strips at room temperature between 15°C–30°C (59°F–86°F). Do not expose the strip to direct sunlight. Ensure the cap is closed tightly after each use.

Important Precautions

Keep the strip in the original container. Remove only the required strip before testing and immediately replace the cap tightly. Exposure of the uncovered strip to light and moisture may cause discoloration of the strip. Do not touch the reagent pad with finger or any other surface.

Product Availability

AHP1750 Indicator Strips are packaged 50 strips per bottle, one (1) bottle per package.

Distributed by/Distribué par :
Virox Technologies Inc.
2770 Coventry Rd., Oakville, ON L6H 6R1
1-800-387-7578 | www.virox.com

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Accelerated Hydrogen Peroxide[®] and Design are trademarks of Diversey, Inc.