

PREempt®

One-Step Disinfectant
Cleaner Wipes formulated
with **AHP**[®]
Patented



Wipes

WHY CHOOSE PREempt™?

- CLEANER** Effective disinfectant as well as a cleaner resulting in added confidence that disinfection can occur.
- FASTER** Rapid and realistic contact times on a broad spectrum of difficult to kill pathogens.
- RESPONSIBLE** Designed to be easier on surfaces, staff and occupants. Users are not exposed to VOCs (volatile organic compounds) and PPE** is not required as per health and safety rating.
- SUSTAINABLE** The active ingredient, Hydrogen Peroxide, breaks down into water and oxygen leaving no active residues which helps to reduce environmental impact.

DIRECTIONS FOR USE

For Use as a One-Step Disinfectant Cleaner Product:

1. Pre-clean heavily soiled areas.
2. Pull towelette from canister and wipe hard, non-porous environmental surfaces.
3. **All surfaces must remain visibly wet for 1 minute.**
Use a 5 minute contact time for TB¹. Use a 10 minute contact time for fungi.
4. Wipe surface dry or allow to air dry.

For Use as a Non-Food Contact Sanitizer:

1. Pre-clean heavily soiled areas.
2. Pull towelette from canister and wipe hard, non-porous environmental surfaces.
3. Allow the surface to remain wet for 30 seconds.
4. Wipe surface dry or allow to air dry.
5. Change towelette after each use.

PREempt™ Surface Disinfectants are designed to protect your staff, patients and business with easy to use disinfectant solutions that are gentle on people, surfaces, and devices.

AVAILABLE PRODUCTS

PREempt™ One-Step Disinfectant Cleaner Wipes

DIN: 74559-3

SHELF LIFE: 3 Years

CONCENTRATION OF ACTIVE:
0.5% Hydrogen Peroxide

** PPE – Personal Protective Equipment
TB¹ stands for BCG (*Mycobacterium bovis*)

EFFECTIVE AGAINST A BROAD-SPECTRUM OF *VIRUSES AND BACTERIA IN 1 MINUTE!

*Virucidal: 1 Min.

- *Poliovirus Type 1
- *Hepatitis B Virus (HBV)
- *Hepatitis C Virus (HCV)
- *Kills HIV-1 (AIDS virus)
- *Kills Norovirus (Feline Calicivirus as the surrogate)
- *Human Coronavirus
- This product has demonstrated effectiveness against Influenza A virus and is expected to inactivate all Influenza A viruses including Pandemic 2009 H1N1 Influenza A virus.

Bactericidal: 1 Min.

- *Pseudomonas aeruginosa*
- *Staphylococcus aureus*
- *Salmonella enterica* (formerly known as *Salmonella choleraesuis*)

Antibiotic-Resistant Bactericidal Activity:

- Methicillin-resistant *Staphylococcus aureus* (MRSA)
- Vancomycin-resistant *Enterococcus faecalis* (VRE)
- *Escherichia coli* with Extended Spectrum Beta-lactamase resistance (ESBL)

Fungicidal: 10 Min.

- *Trichophyton mentagrophytes*

Tuberculocidal: 5 Min.

- †*Mycobacterium bovis* (BCG)

Broad-Spectrum Non-Food Contact Sanitizing: 30 Sec.

- *Klebsiella pneumoniae*
- *Salmonella enterica* (formerly known as *Salmonella choleraesuis*)
- *Pseudomonas aeruginosa*
- *Staphylococcus aureus*
- *Escherichia coli* O157:H7



PREempt™ One-Step Disinfectant Cleaner Wipes are made of nonwoven, meltblown polypropylene wipe material, ensuring even surface coverage and increasing the ability and efficiency of AHP® for fine particle removal.

*Please refer to reference sheet for a complete list of pathogenic organisms, additional features, and claims.

PREempt™ brand products are manufactured by Virox Technologies Inc.

Accelerated Hydrogen Peroxide® (AHP®) and PREempt™ are registered trademarks of Virox Technologies Inc.



For more information on this or our other products, visit:

> virox.com

2770 Coventry Road, Oakville, ON L6H 6R1 CANADA
1-800-387-7578 | www.virox.com



Engineering Revolutionary Disinfectants for the War Against Microbes